



How to Download & Install App in your Phone

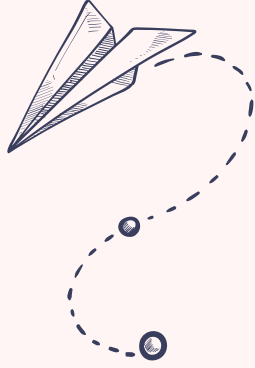


Follow these easy steps to get started.

 <https://vitalytiqai.com>

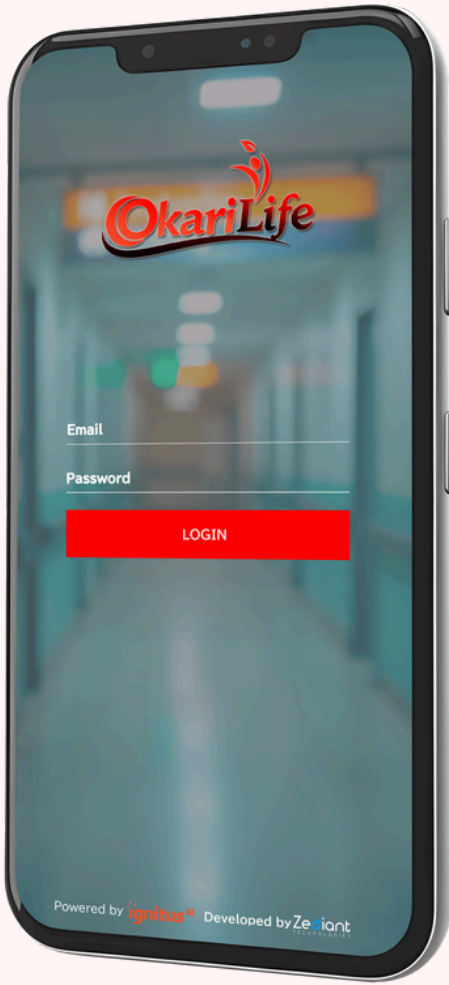


Scan the QR code to go
directly to the website:

<https://vitalytiqai.com/app-okarilife>

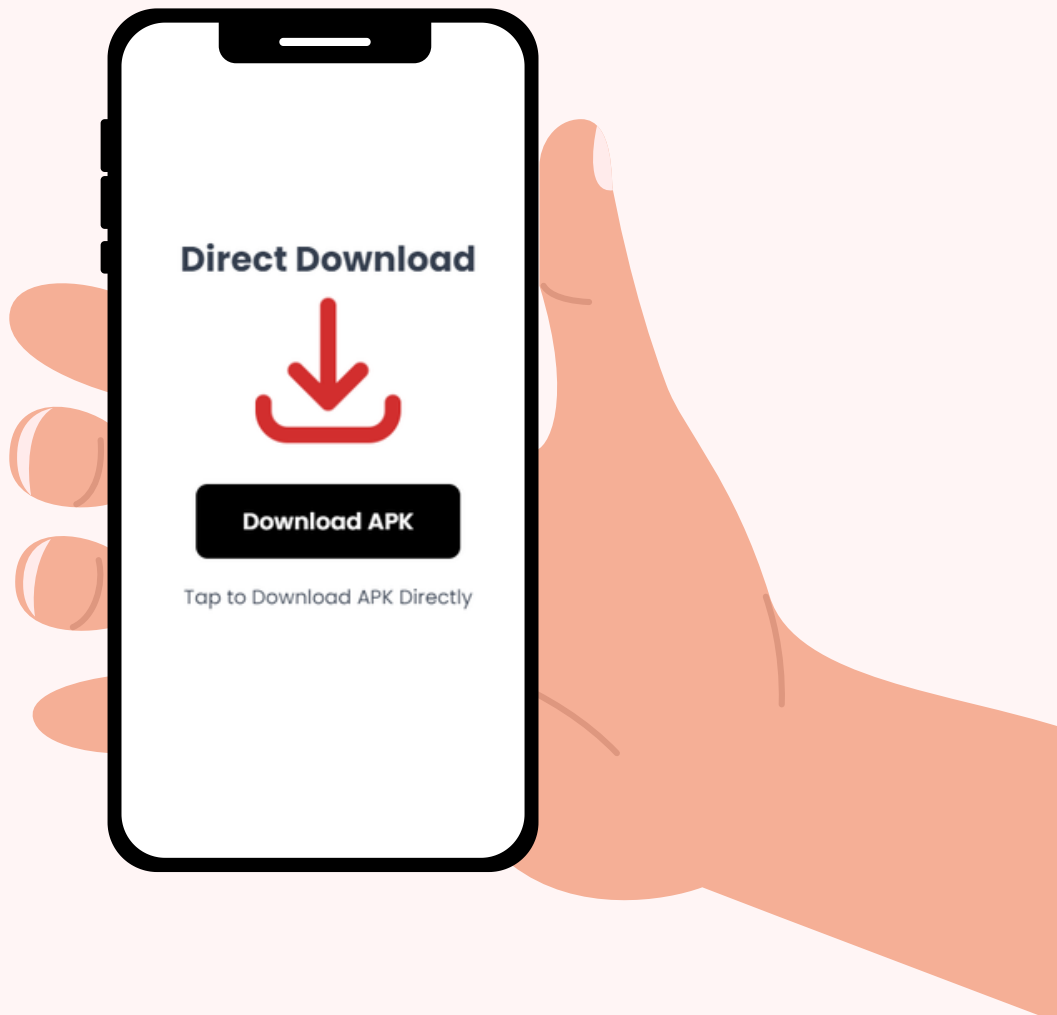

Launch Installer


After clicking on this Link , you have to install the package installer. Click on the download icon

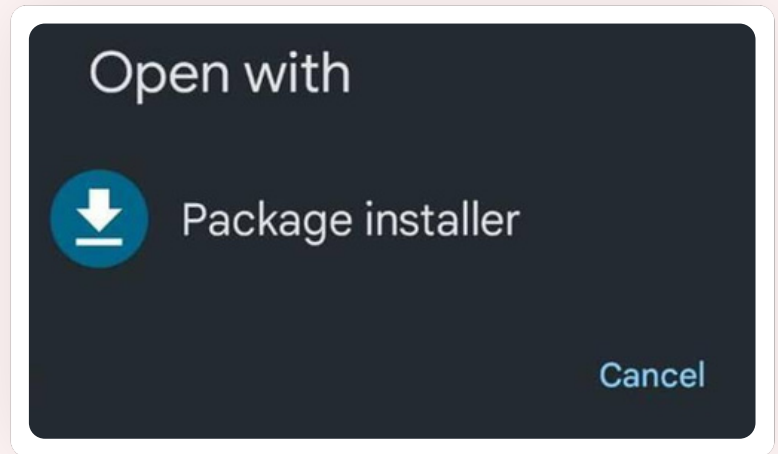

Step 2

Click On Download:


Click the "Download" button to get the installer to your device.

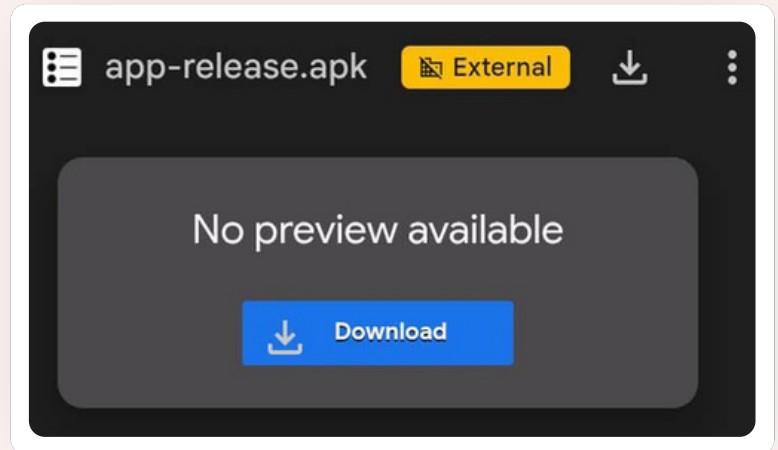


Step 3

Open: app-release.apk
(downloaded file)

Click on: Install

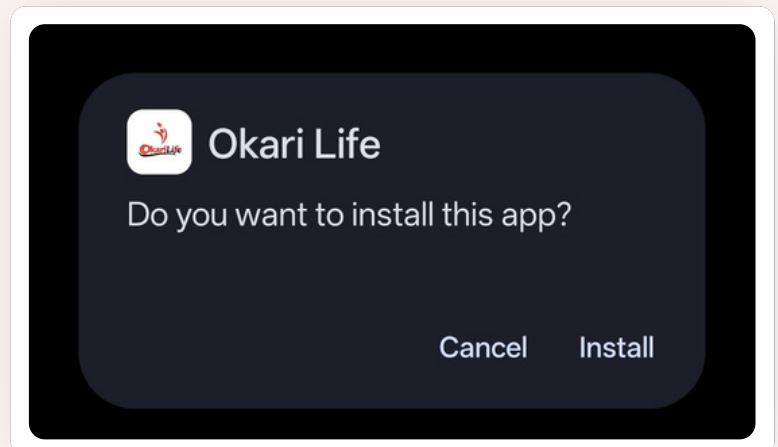

Step 4

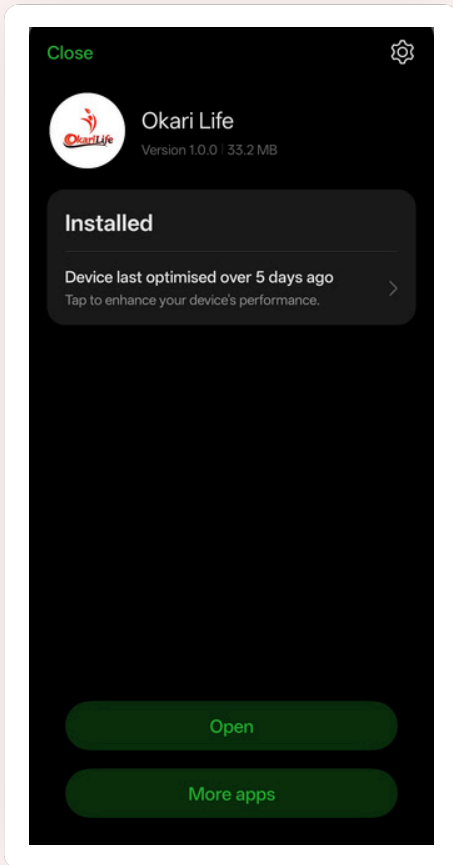




After passing all Device Safety Tests


Click on: Open button

Step 5

Step 6

Allow Location and Call permissions as requested. The options may vary depending on the device, so please read the instructions in the popup carefully while installing the app

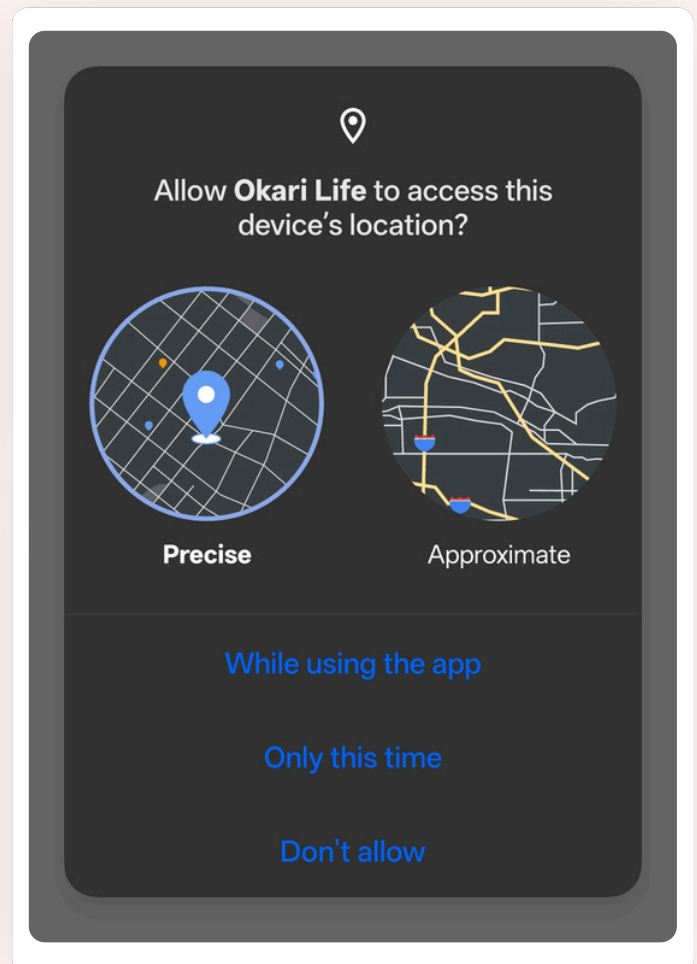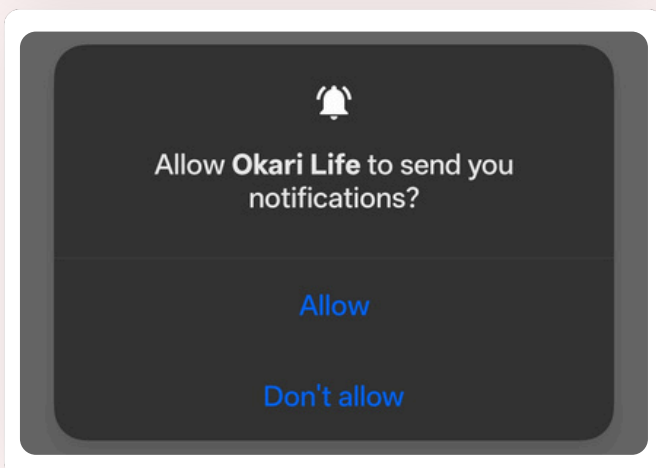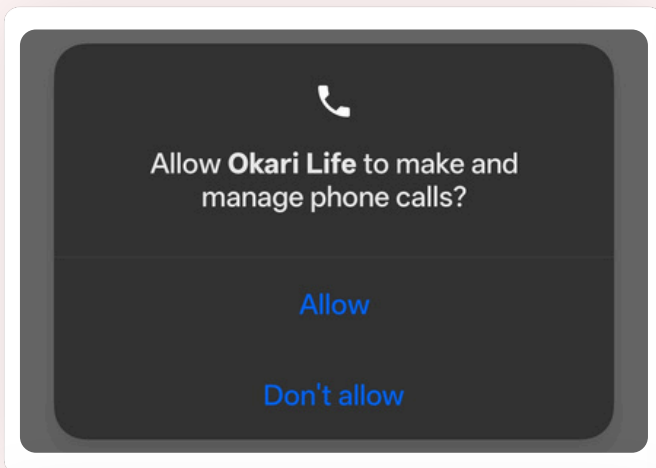

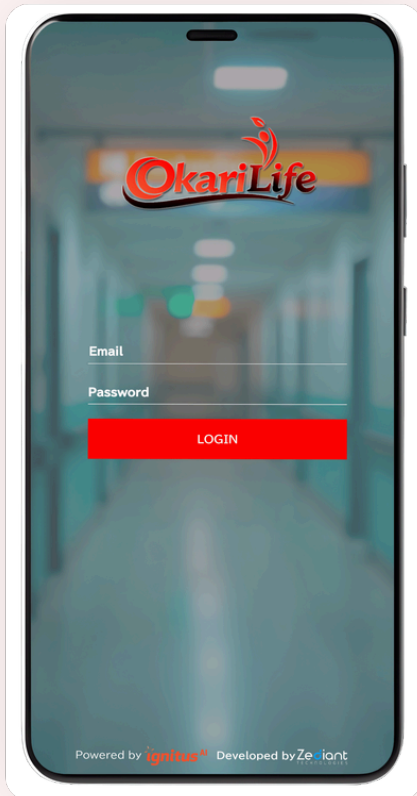




After installing the OkariLife app, enter your email and password to log in

Step 7

Step 8




Read the 'Permission Required' popup carefully and follow the guided steps (For Xiaomi devices, the following options appear)

1. "Go to settings"
2. Click on the "App Permissions"
3. Allow the two options marked with a red cross:
 - Display pop-up windows while running in the background
 - Display pop-up window

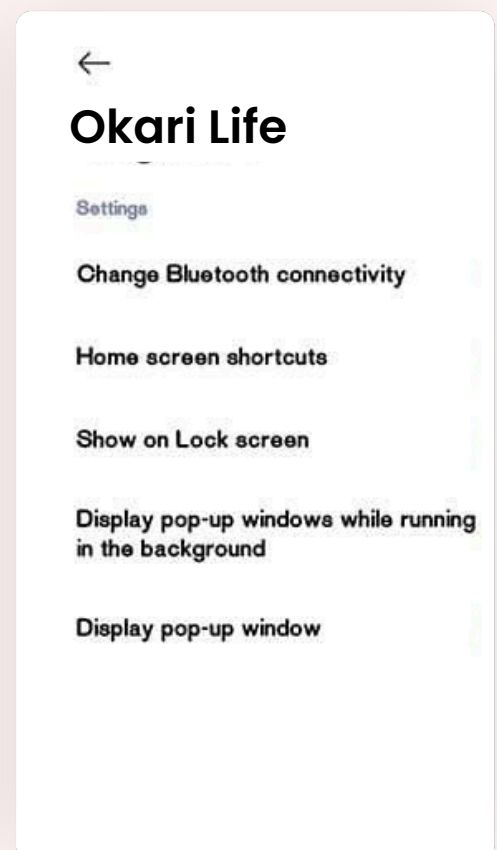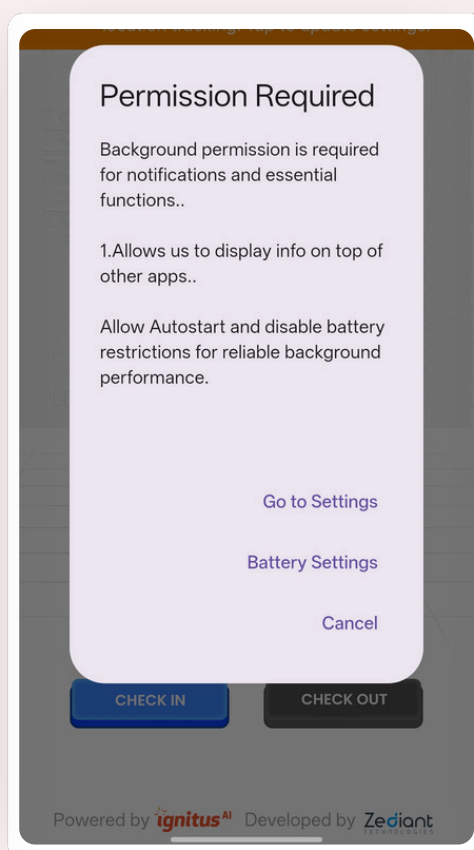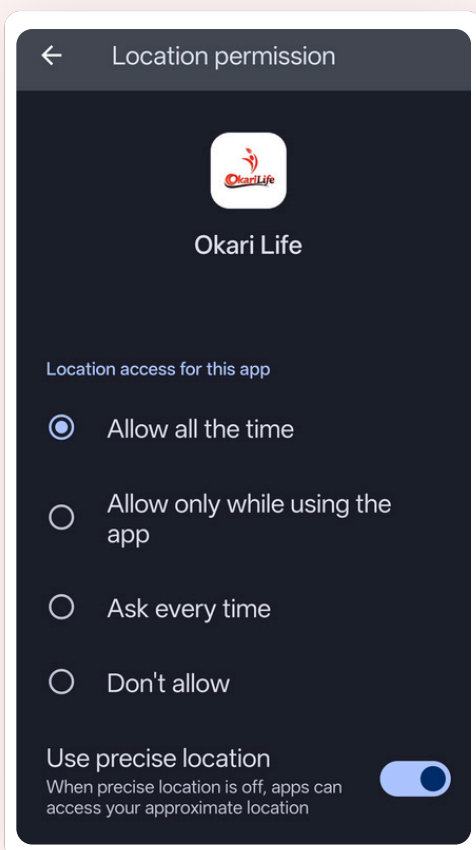

For other devices

- 1.Go to settings
- 2.Click on display over other apps
- 3.Enable the "Display over other apps" option

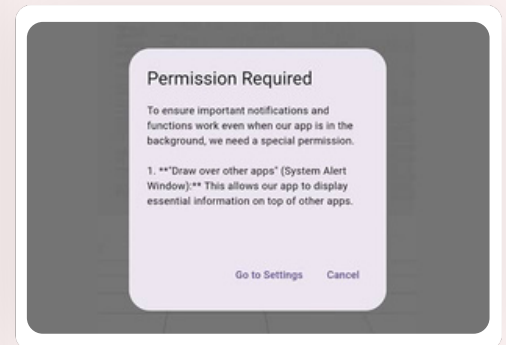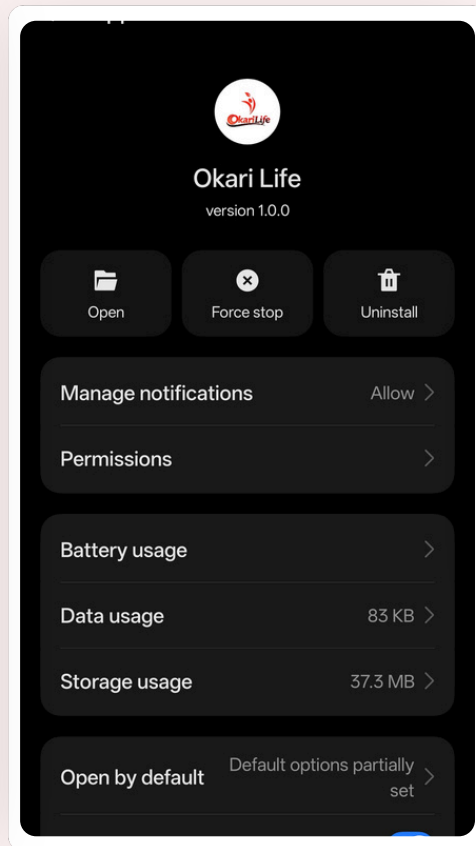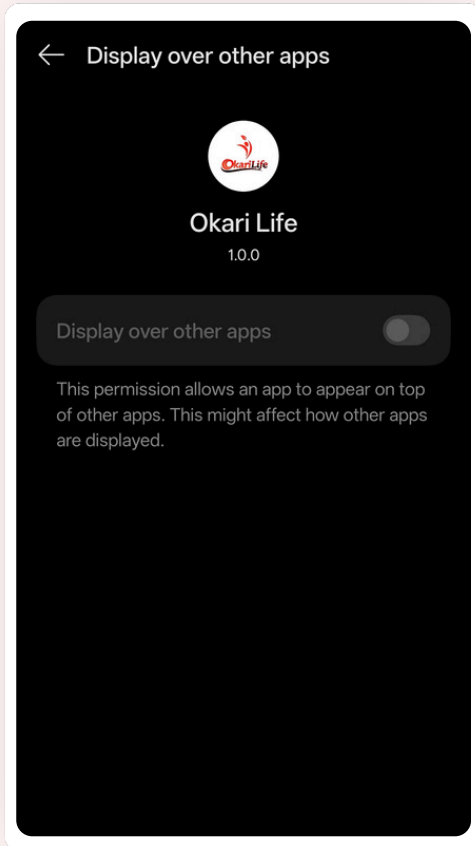


Step 9

After allowing the necessary options, go back to the home screen.

Click on the note below the header area to allow Location Permission as 'Allow all the time' from mobile settings.



Now the app is ready to be used

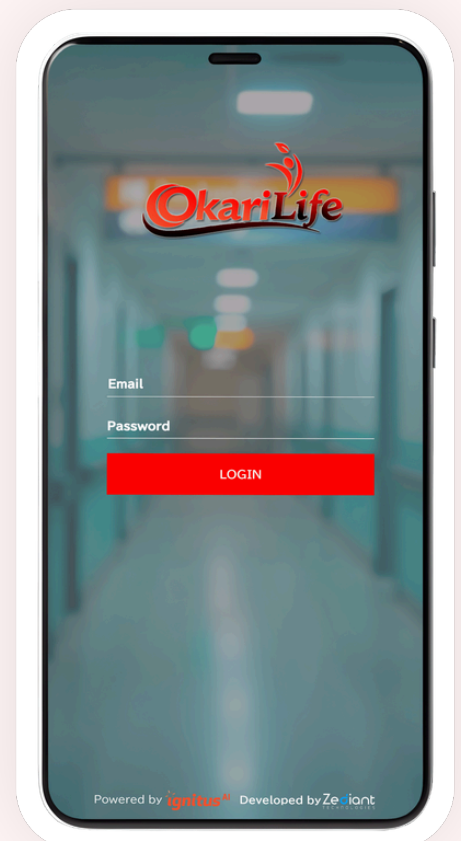

Health at Your Fingertips

Check your BP or Heart Rate with a simple tap.

Reports sent to your mobile widget instantly.



Tap



Measure



Receive

How it Works

Simple steps to better health



TAP

Simply tap on your watch to start



MEASURE

Your watch checks your health rate



RECEIVE

Your report is sent to your mobile widget instantly

Your Health. Your Data. Your Control.

Secure, private and always with you.



Secure Data



Private & Confidential



Instant Results




Setup Mobile Widget

[Get Started](#)

Let's check the quick setup for the mobile widget.




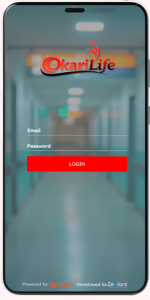

Install & Login

Install the app from the link & login with your credentials

xxxxxx
xxxxxx
xxxxxx
xxxxxx

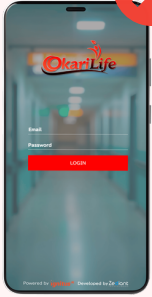

Close App

After successful Login close the app & go to the home screen



Press & Hold

On an empty area of your phone home screen



Select App

A menu will appear at the footer & select the widgets option



Find Okarilife App

In the alphabetical list



Widget Placed

The widget will appear on your main home screen

xxxxxx
xxxxxx
xxxxxx

THANK YOU

GET IN TOUCH WITH US



Contact us via WhatsApp

(+61) 424 027 909



Website:

www.vitalytiqai.com

