

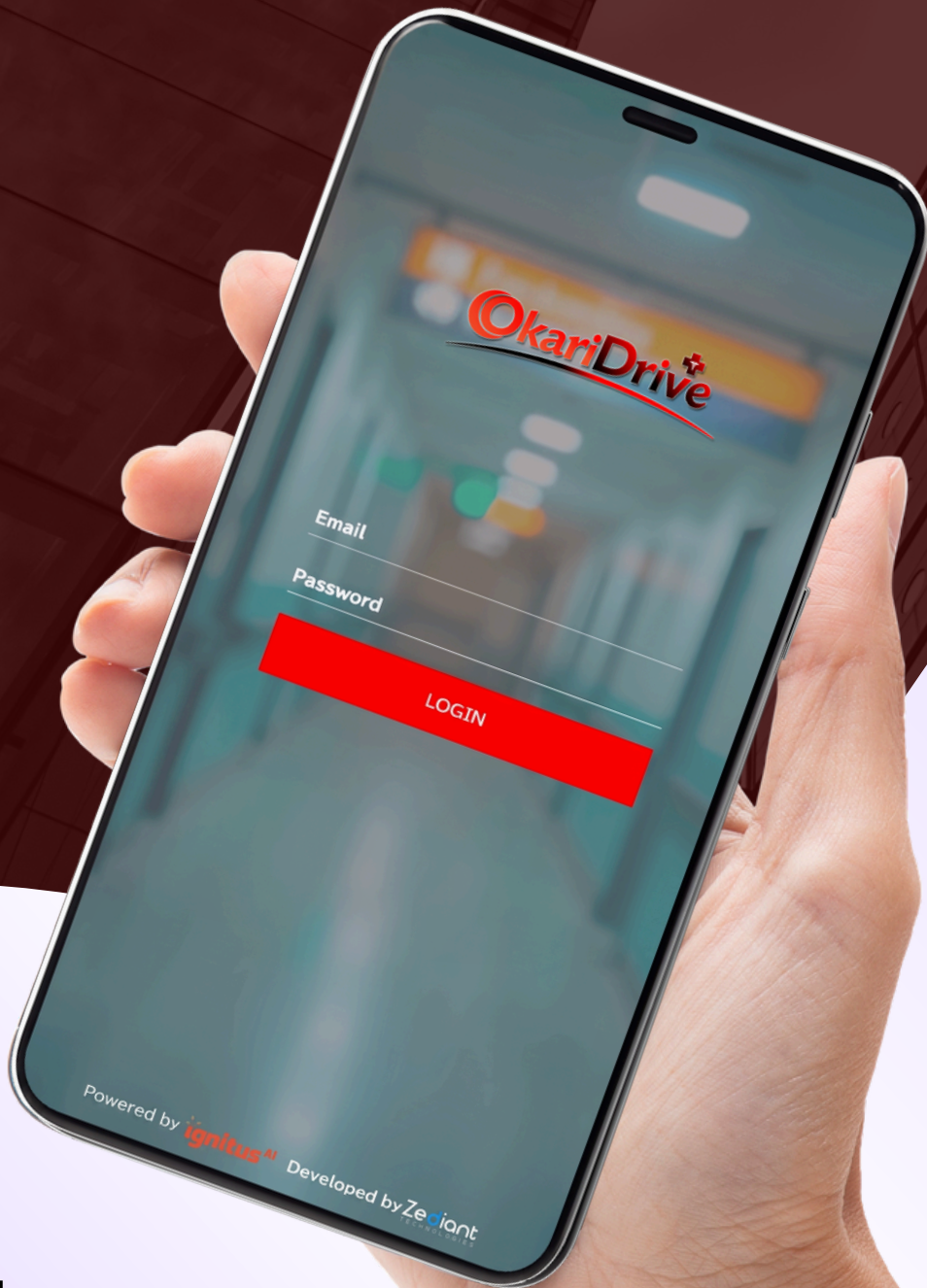


# How to Download & Install App in your Phone

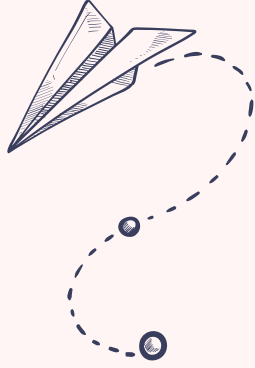


Follow these easy steps to get started.

 <https://vitalytiqai.com>

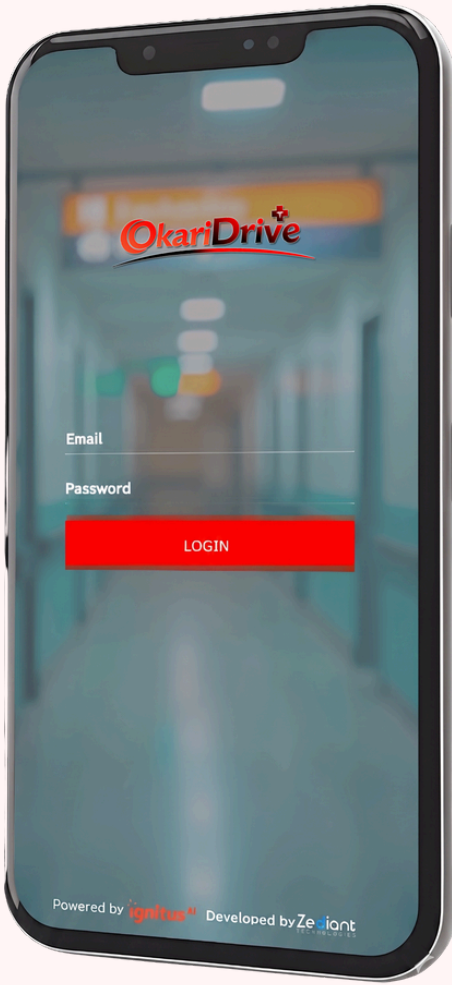

Scan the QR code to go  
directly to the website:


<https://vitalytiqai.com/app-okaridrive>

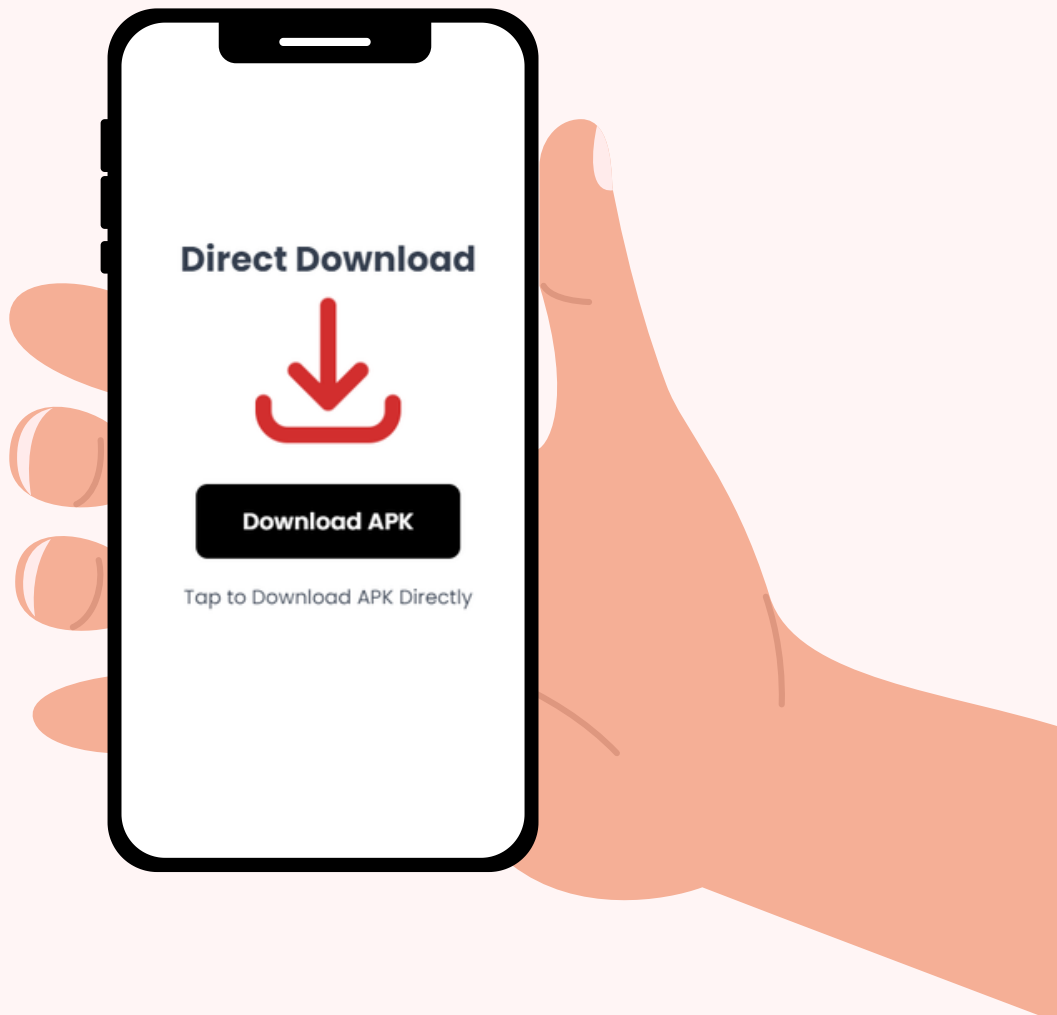

Step 1

## Launch Installer


After clicking on this Link , you have to install the package installer. Click on the download icon

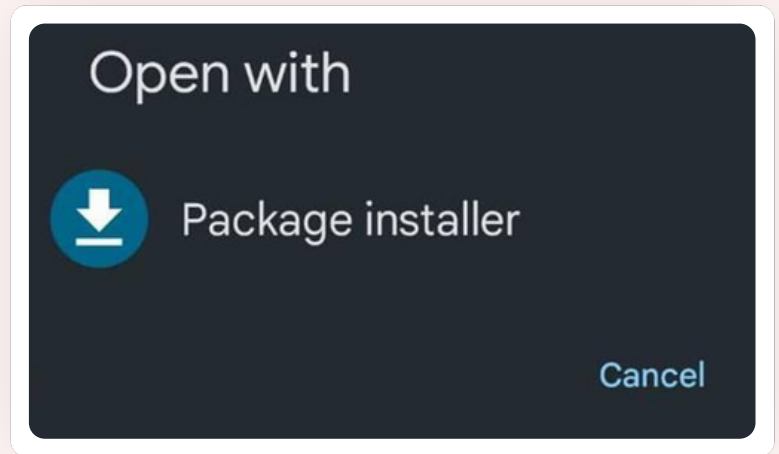

## Step 2

## Click On Download:


Click the "Download" button to get the installer to your device.

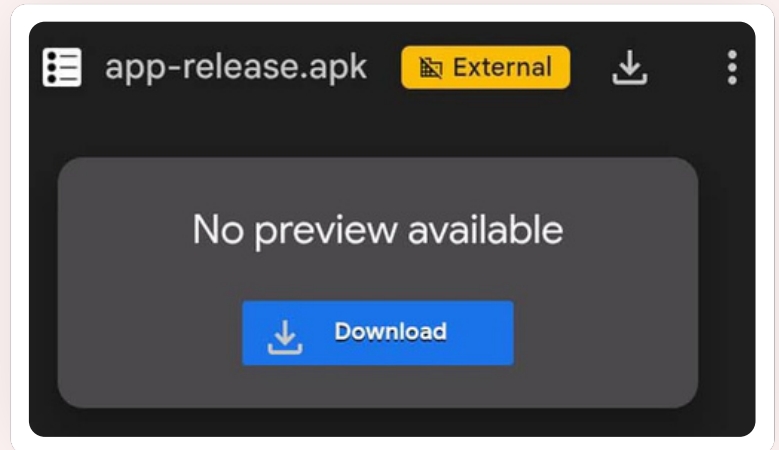


## Step 3

**Open:** app-release.apk  
(downloaded file)

**Click on:** Install

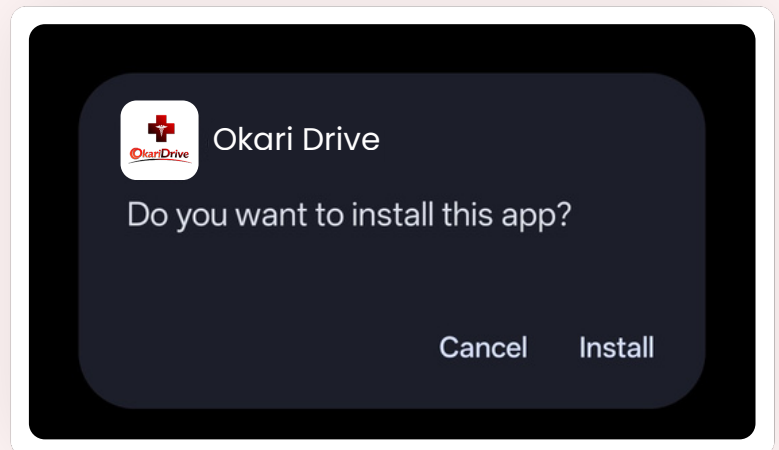

## Step 4

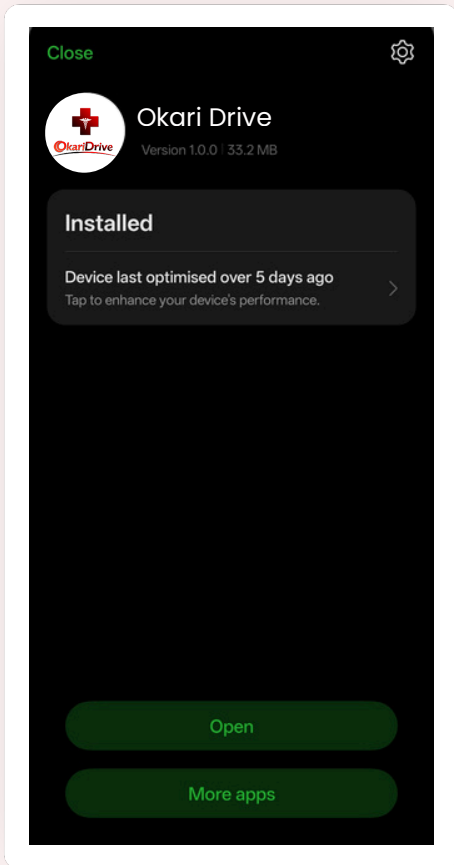




After passing all Device Safety Tests


**Click on:** Open button

## Step 5

## Step 6

Allow Location and Call permissions as requested. The options may vary depending on the device, so please read the instructions in the popup carefully while installing the app

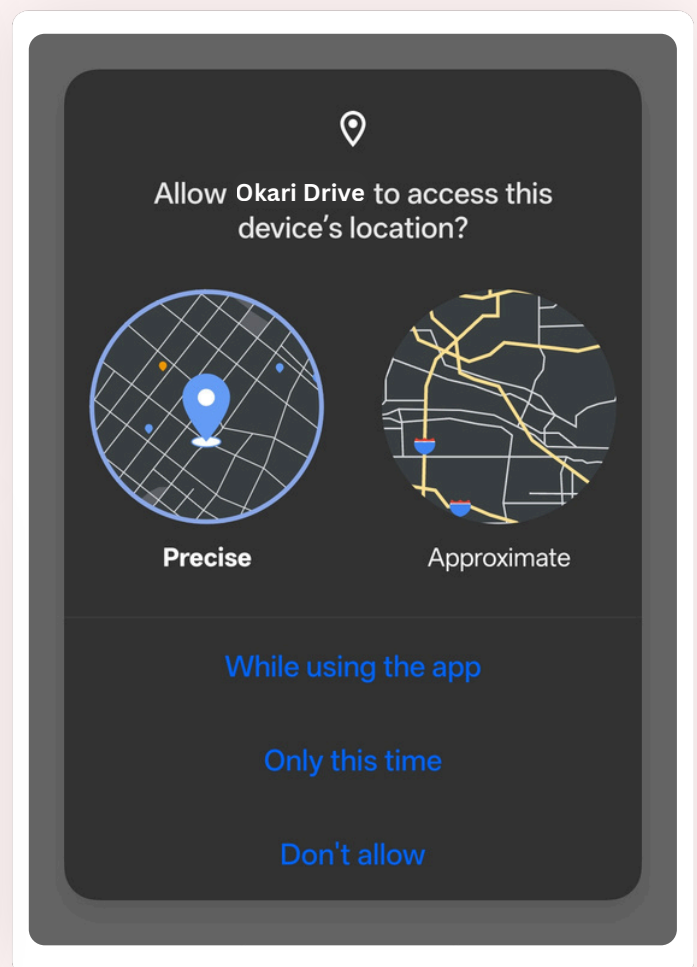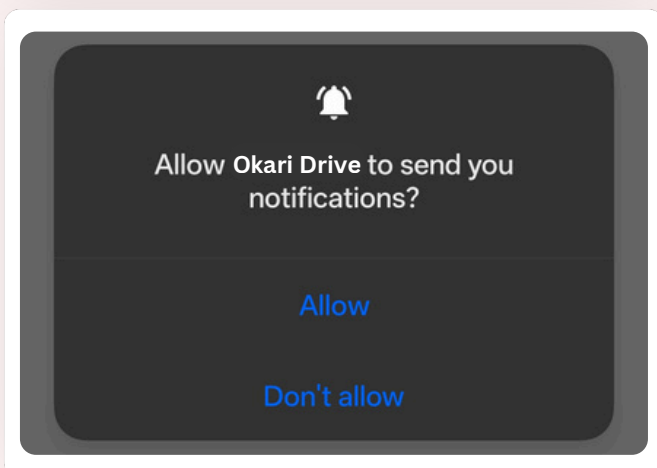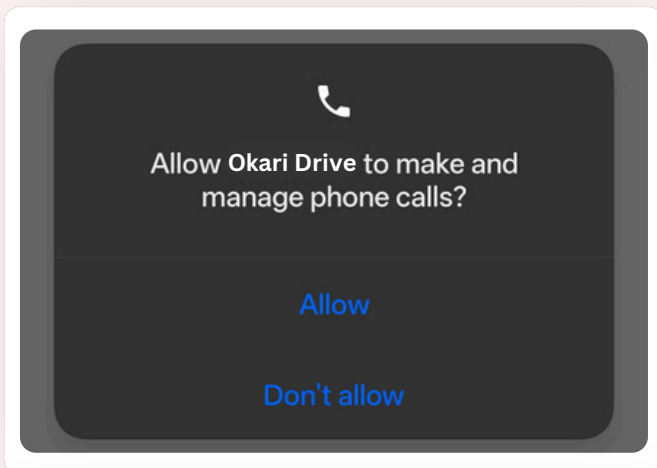

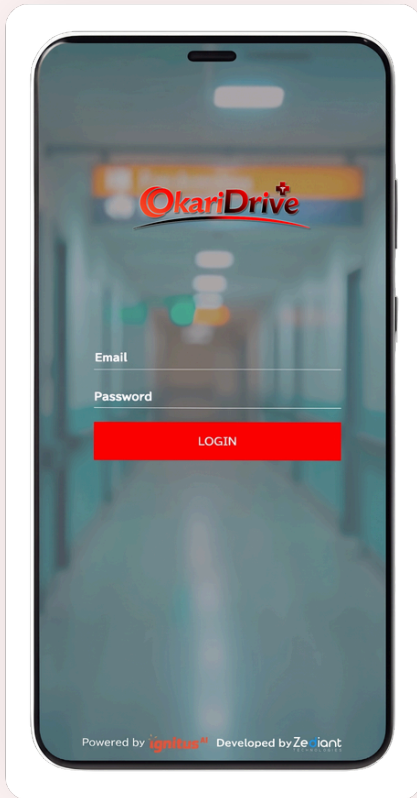




After installing the OkariDrive app, enter your email and password to log in

## Step 7

## Step 8




Read the 'Permission Required' popup carefully and follow the guided steps (For Xiaomi devices, the following options appear)

1. "Go to settings"
2. Click on the "App Permissions"
3. Allow the two options marked with a red cross:
  - Display pop-up windows while running in the background
  - Display pop-up window

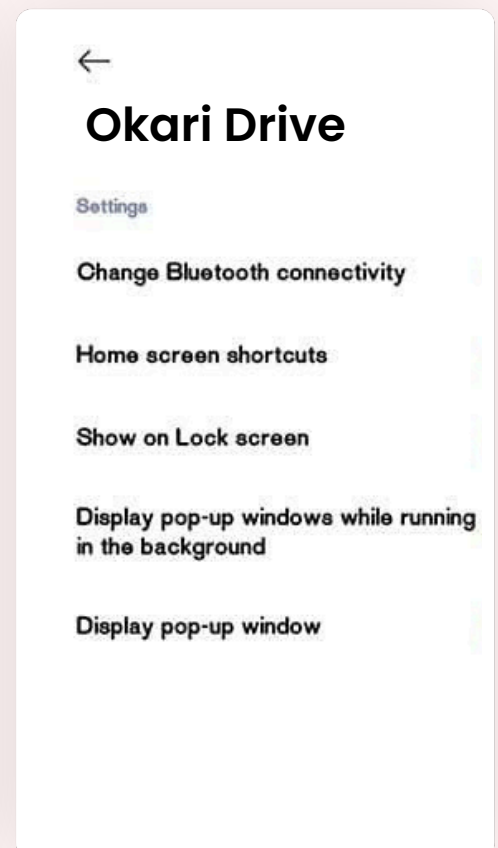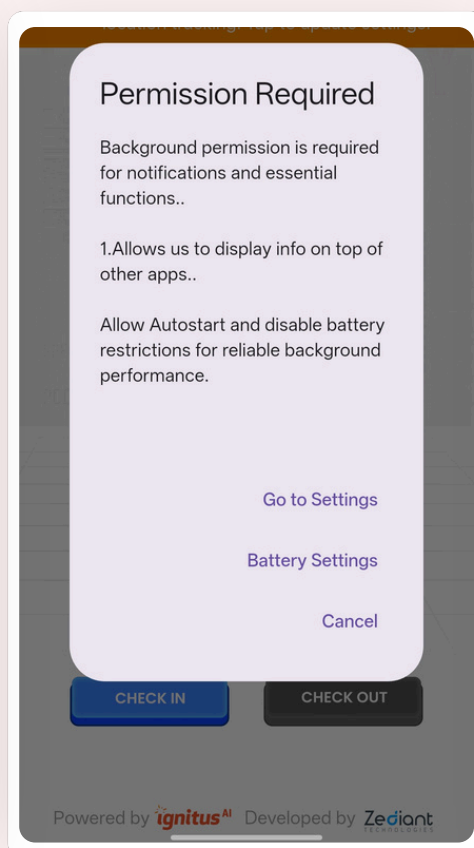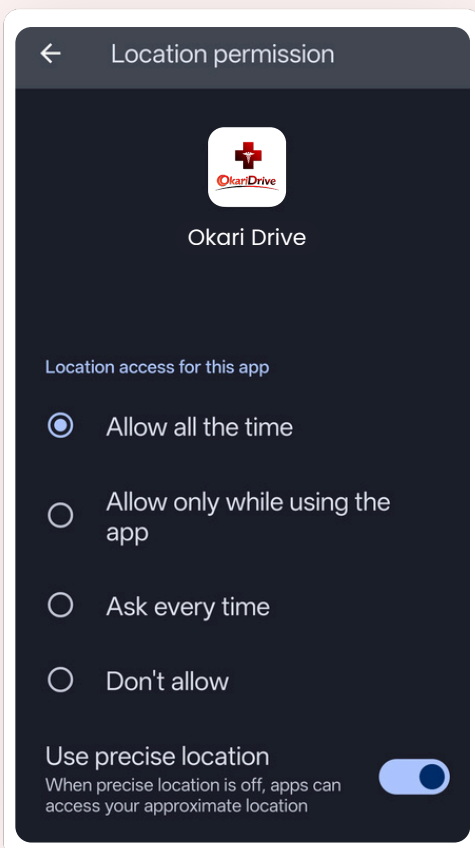

For other devices

1. Go to settings
2. Click on display over other apps
3. Enable the "Display over other apps" option

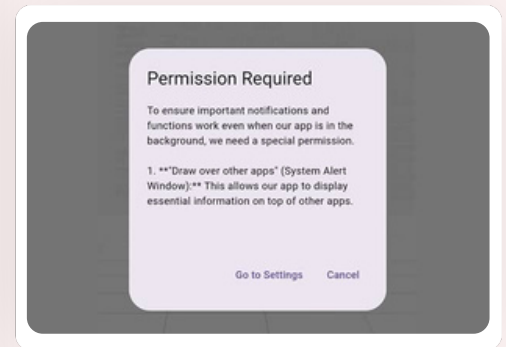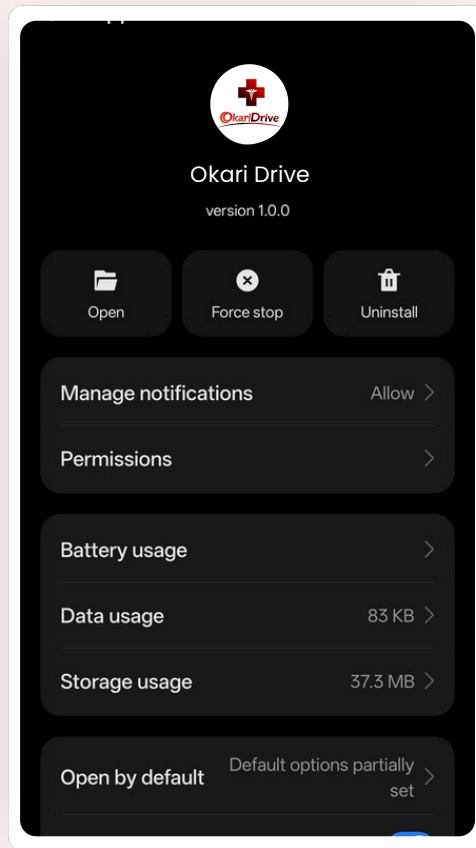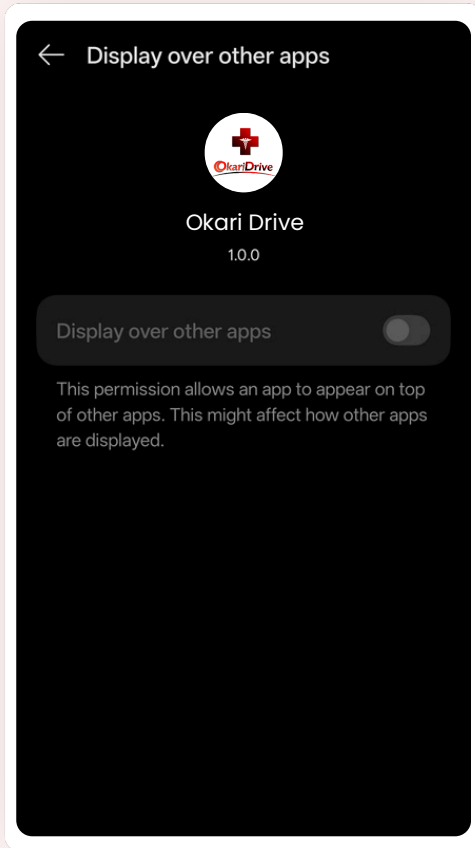


## Step 9

After allowing the necessary options, go back to the home screen.

Click on the note below the header area to allow Location Permission as 'Allow all the time' from mobile settings.



Now the app is ready to be used

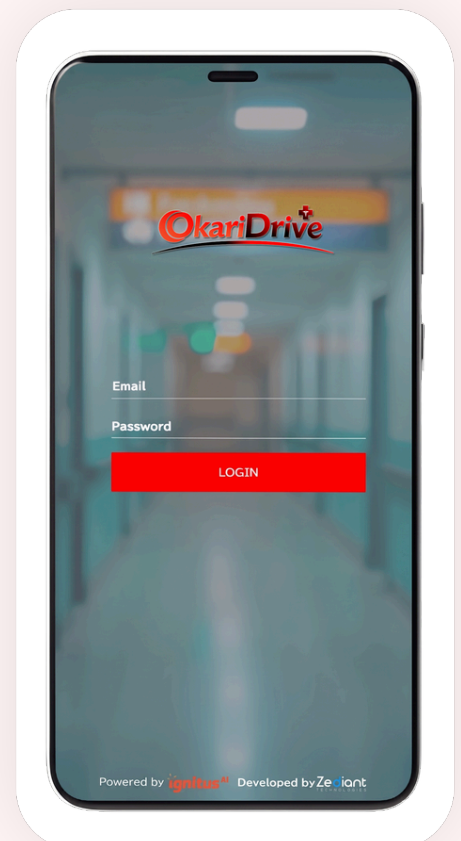

# How OkariDrive Works?

Monitors driving behavior and patient stability, combining WH&S safety and telematics data to deliver real-time monitoring and emergency response.



01

Install the  
OkariDrive  
Device



02

Data  
Captured  
During  
Every Drive



03

Alerts &  
Reports  
Generated



# ..... Various Features .....



**SOS – Duress Button**  
Instant SOS support for immediate help during emergencies.

**Guardian Angel**  
Timed check-ins with automatic SOS alerts.

**Self or Professional Monitoring**  
24/7 self or professional monitoring.

**Driver Behaviour Tracking**  
Driver behavior monitoring with critical insights.

**Exception Reporting**  
Automatic alerts for behavior exceptions.

**Full Timeline History**  
Detailed logs for long-term care insights.

**Driver and Fleet Reports**  
Detailed driver and fleet reports.

**Smart Monitoring for Safer Patient Mobility**  
One application, one visualisation

  
Predictive Alerting

  
Ecosystem Integration

  
Learned Normal Profile

# THANK YOU

## GET IN TOUCH WITH US



**Contact us via WhatsApp**

(+61) 424 027 909



**Website:**

[www.vitalytiqai.com](http://www.vitalytiqai.com)

