



# How to Download & Install App in your Phone

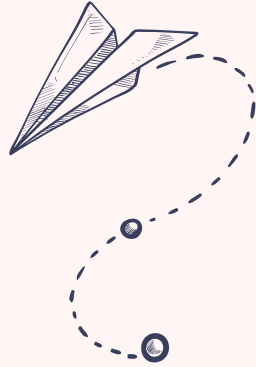


Follow these easy steps to get started.

 <https://vitalytiqai.com>

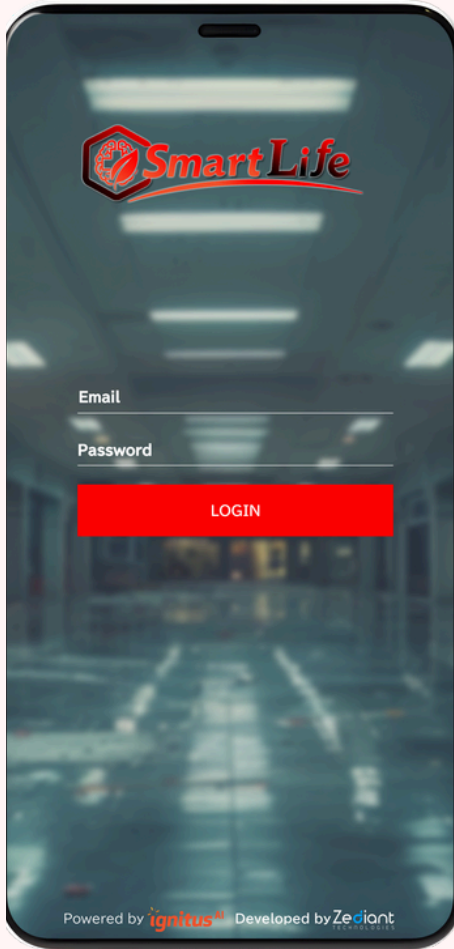


Scan the QR code to go  
directly to the website:

<https://vitalytiqai.com/app-smartlife>

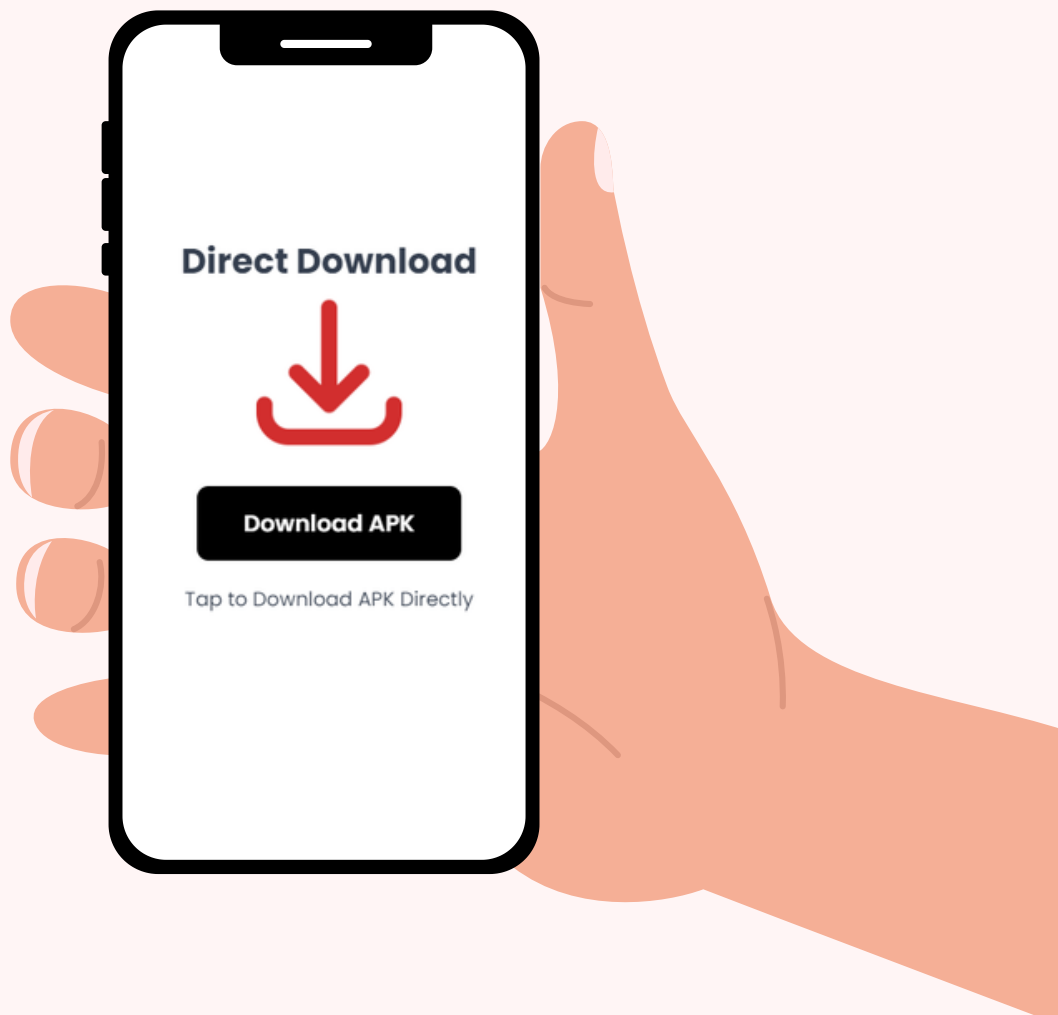

## Launch Installer


After clicking on this Link , you have to install the package installer. Click on the download icon

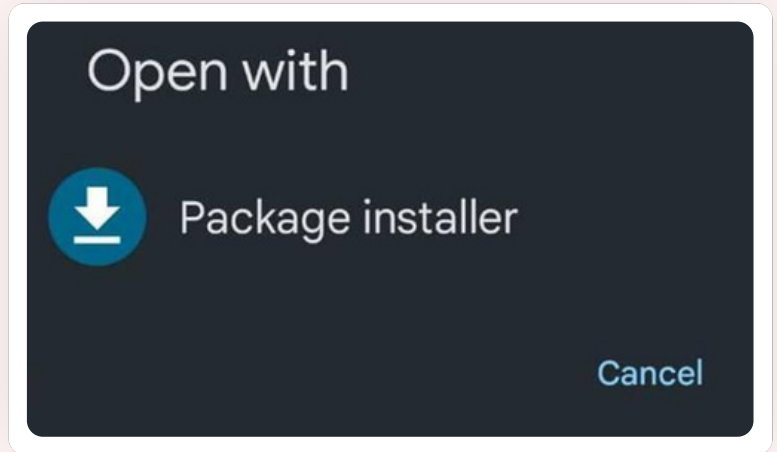

## Step 2

## Click On Download:


Click the "Download" button to get the installer to your device.

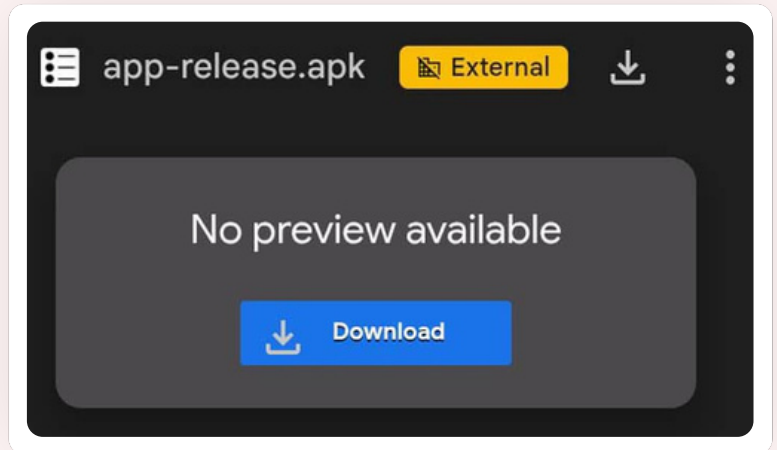


## Step 3

**Open:** app-release.apk  
(downloaded file)

**Click on:** Install

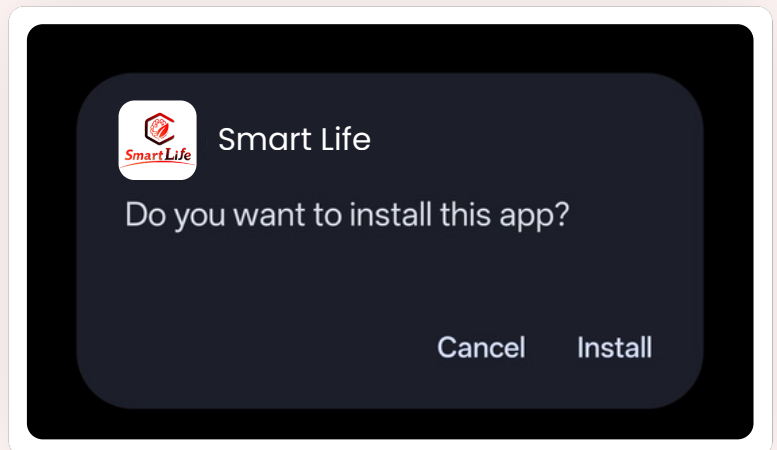

## Step 4

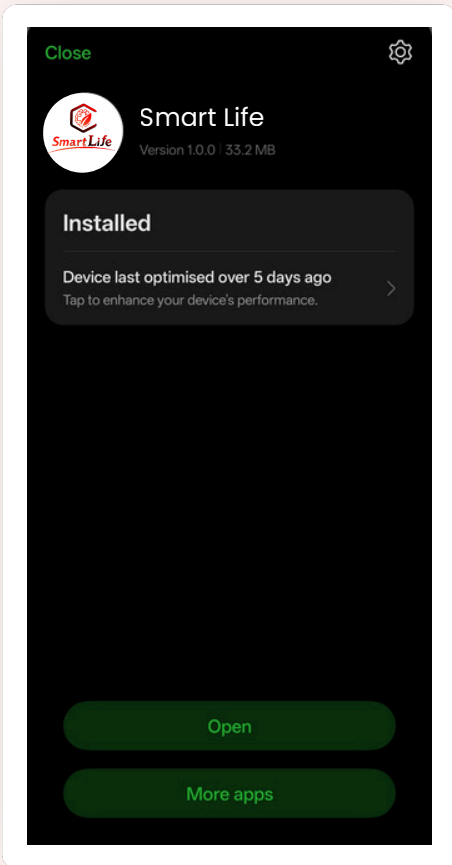




After passing all Device Safety Tests


**Click on:** Open button

## Step 5

## Step 6

Allow Location and Call permissions as requested. The options may vary depending on the device, so please read the instructions in the popup carefully while installing the app

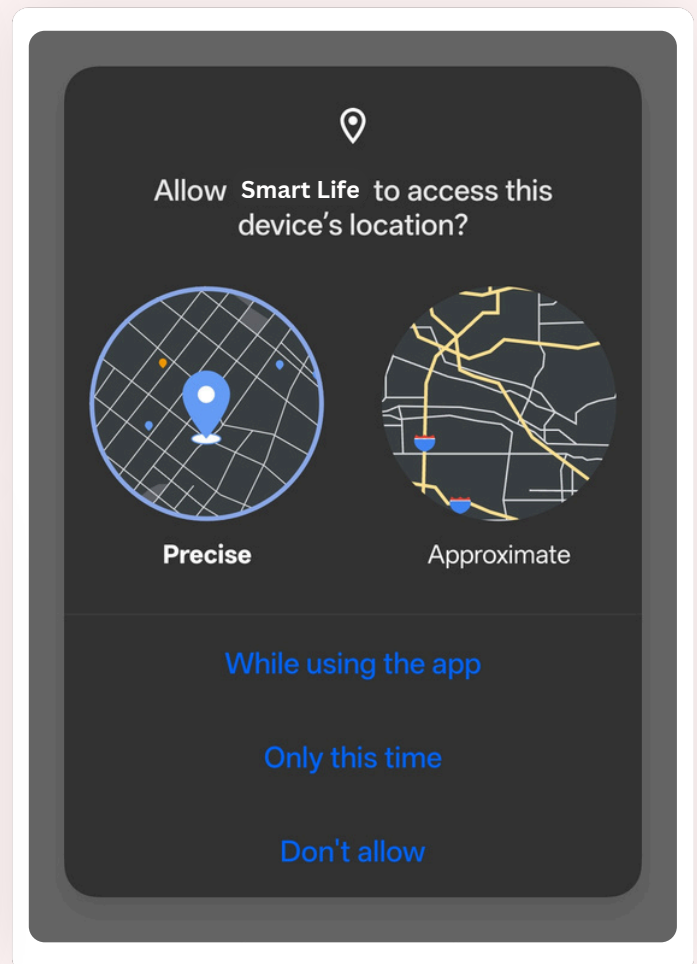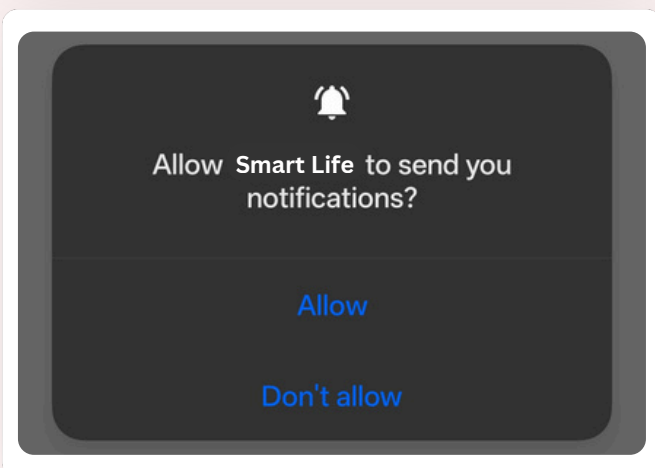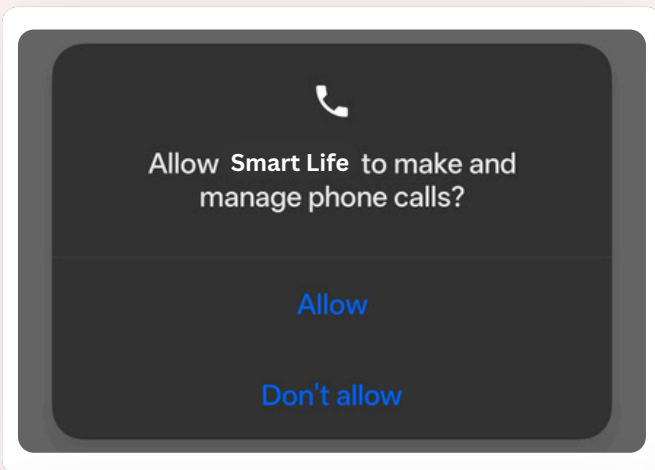

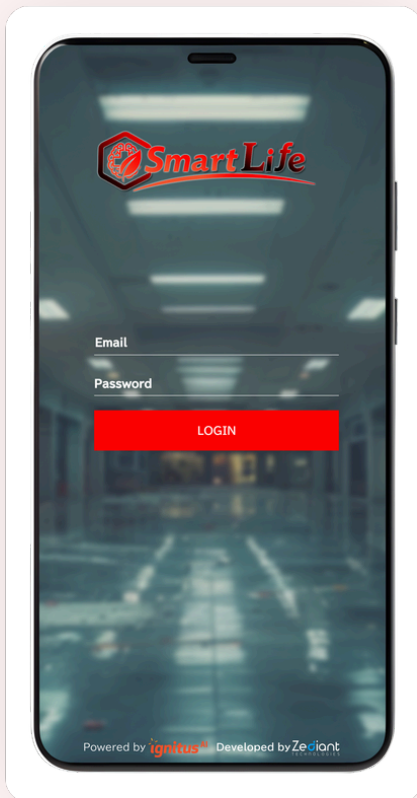




After installing the SmartLife app, enter your email and password to log in

## Step 7

## Step 8




Read the 'Permission Required' popup carefully and follow the guided steps (For Xiaomi devices, the following options appear)

1. "Go to settings"
2. Click on the "App Permissions"
3. Allow the two options marked with a red cross:
  - Display pop-up windows while running in the background
  - Display pop-up window

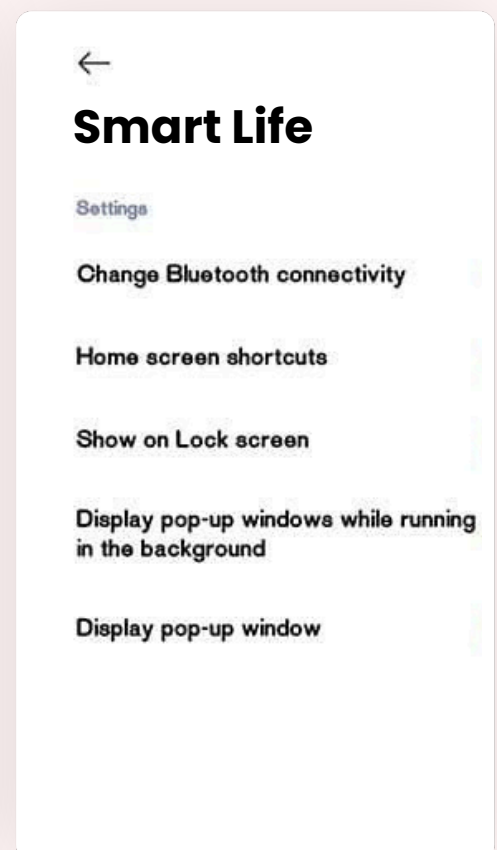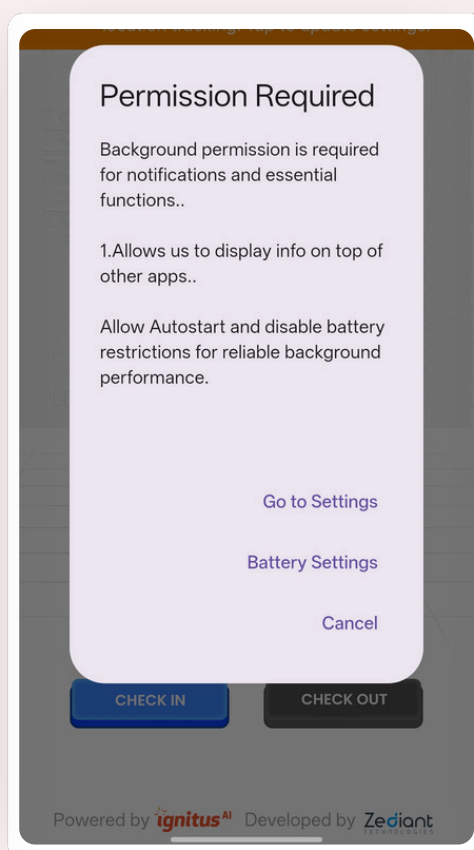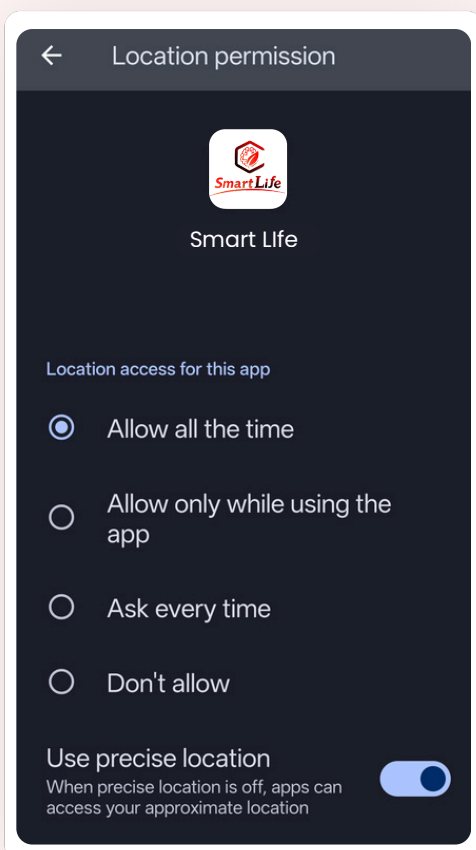

For other devices

- 1.Go to settings
- 2.Click on display over other apps
- 3.Enable the "Display over other apps" option

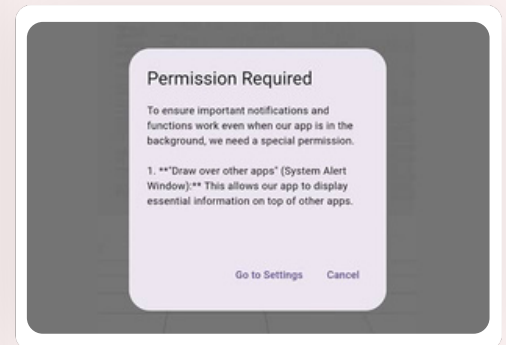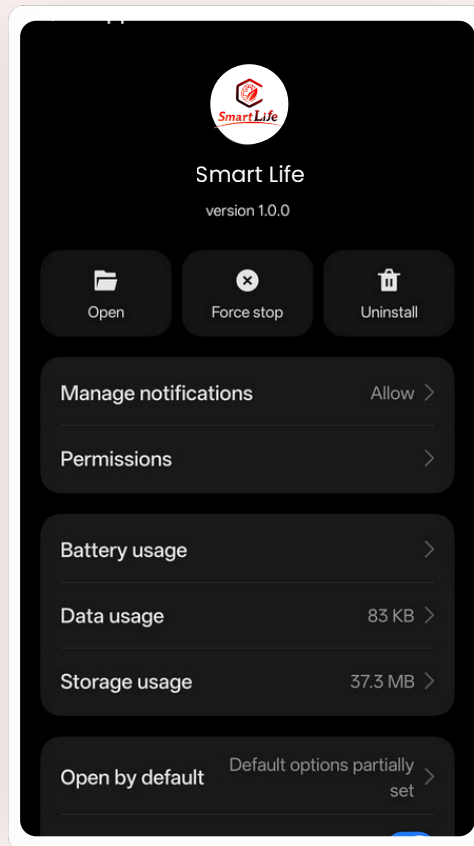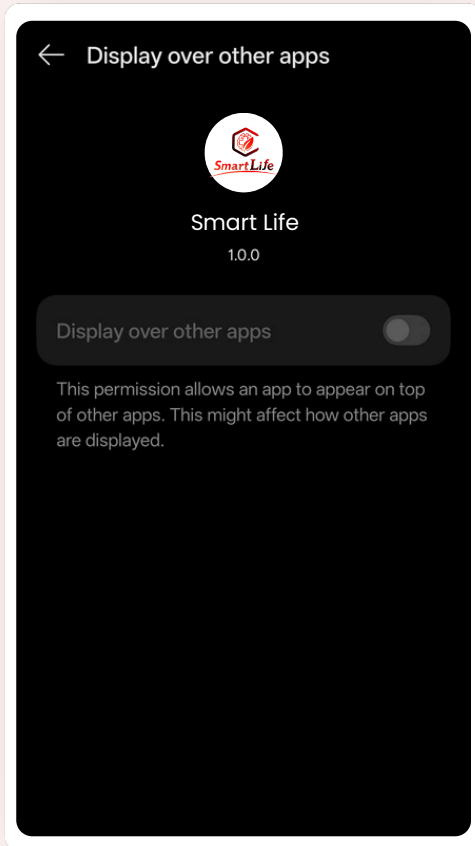


## Step 9

After allowing the necessary options, go back to the home screen.

Click on the note below the header area to allow Location Permission as 'Allow all the time' from mobile settings.



Now the app is ready to be used

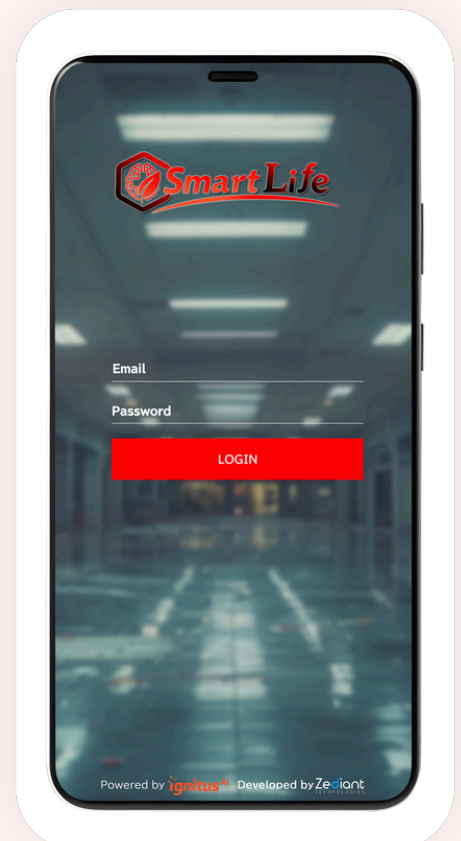

# How **SmartLife** Works?

Everyone's health journey is unique. Explore our full range of easy-to-use smartwatches to find the perfect fit for your lifestyle and safety needs.



01

**Smart  
Devices  
Installed**



02

**Continuous  
AI  
Monitoring**




03

**IgnitusAI  
Cloud  
Intelligence**



# ..... Various Features .....

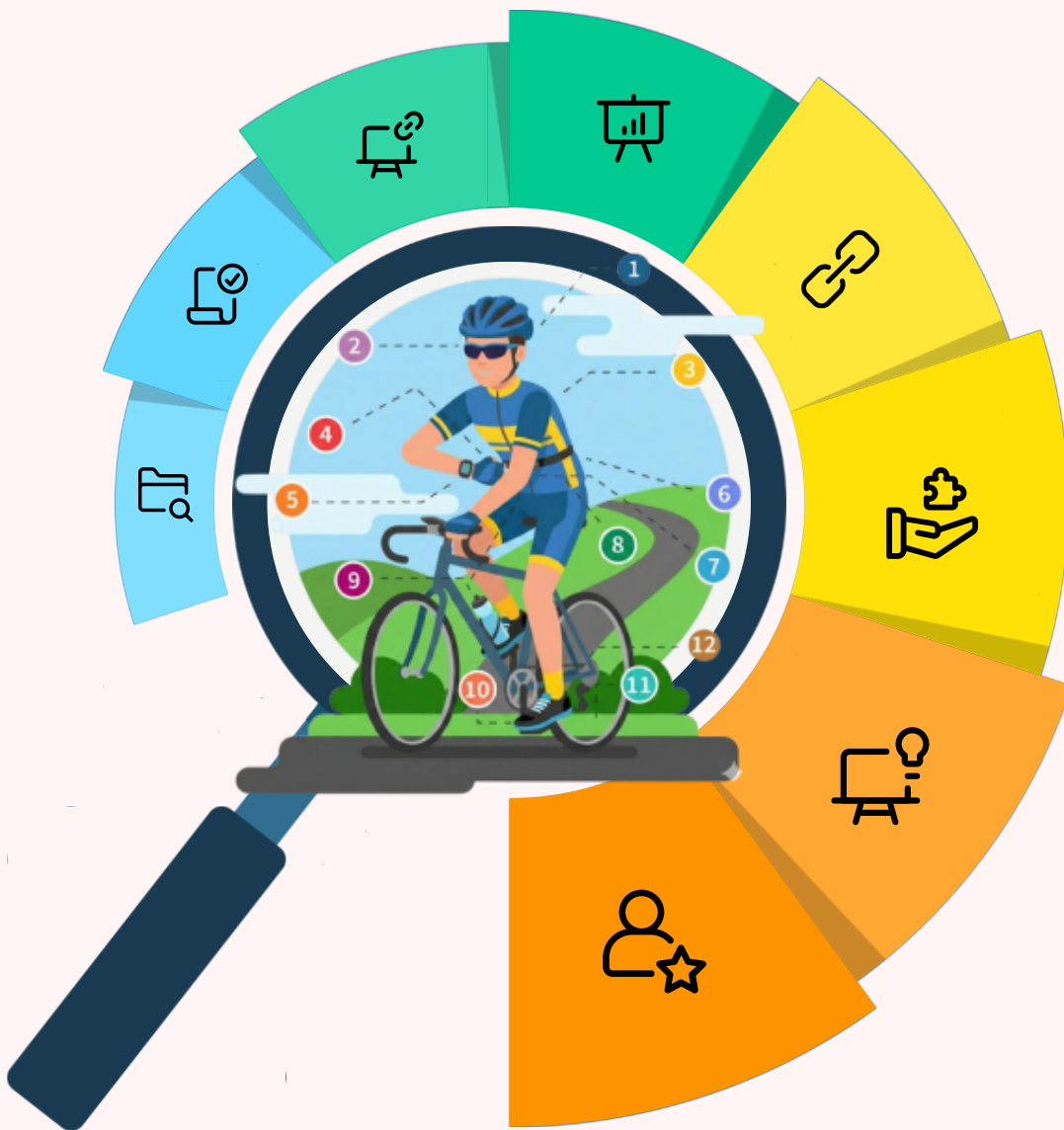

**24x7 AI Health & Behaviour Monitoring**  
Tracks vitals, movement, falls, sleep, mood, and wellness.

**Smart Safe House Automation**  
Smart devices work together for safer living.

**Multi-Device Integration**  
Connects with smart devices and medical IoT.

**Virtual Care Dashboard**  
Caregivers get live updates and safety insights.

**IgnitusAI Cloud Connectivity**  
All data syncs to one smart cloud platform.

**Independent Smart Living (ISML) Model**  
Safe, monitored support for seniors and high-risk users.

**Supports NDIs & Mental Health Services**  
Supports NDIS, reintegration, and complex care.

**Real-Time Alerts & Crisis Prevention**  
The Future of Safe, Independent Living

Predictive Alerting

Optimized Sharing

Secure Data Store

# THANK YOU

## GET IN TOUCH WITH US



**Contact us via WhatsApp**

(+61) 424 027 909



**Website:**

[www.vitalytiqai.com](http://www.vitalytiqai.com)

